

BNG® COVID-19 Antigen Rapid Test Cassette

(Saliva)

Quick Reference Guide

NOTE:

Each test is disposable. Do not try to use the test more than once.


TEST PROCEDURE




1) Take out the test cassette, saliva collector and extraction solution with the blue cap from the box.




2) Wear the gloves. Use a new glove for each patient.

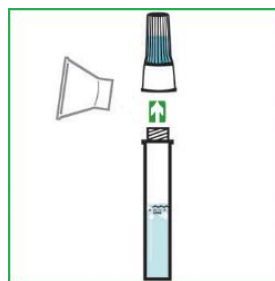


3) Open the cap of the blue tube containing the extraction solution and insert the saliva collection device.

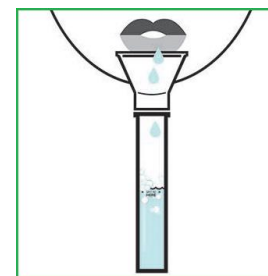

4) Spit out so that it does not cross the transparent line on the tube.

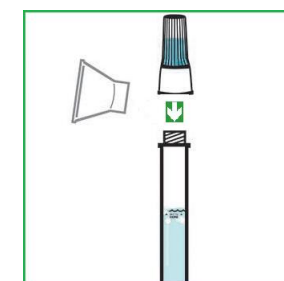

5) Remove the saliva collection device and cap the extraction tube.

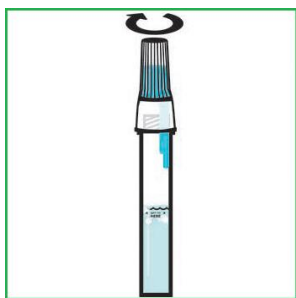


6) Shake the tube more than three times to mix the saliva and extraction buffer.




7) Tear off the tab indicated in the green circle at the top of the blue cap.




8) Add 3 drops to the sample well (S) of the test cassette. Read the result after 15 minutes. Do not interpret the result after 30 minutes.

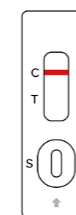

Negative

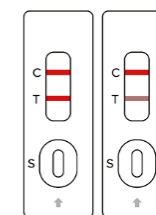

Positive

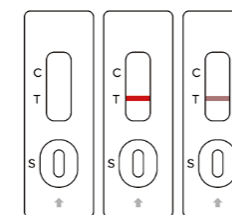

Invalid

Positive: Two lines should be identified. One of the colored lines should be in the control line region (C) and the other line should be in the test region (T).

Negative: Only the control wise (C) should have a single colored line. No lines should be visible in the (T) region.

Invalid: If the control line (C) is not visible. the test is invalid.

NOTE: Before performing Step 4, please do not eat, drink, smoke, vape or brush your teeth within 30 minutes for best results. Do not use mouthwash 2 hours before collecting samples.

TO INCREASE SPECIMEN SENSITIVITY, TAKE A NASAL SAMPLE NOT EXCEEDING 5 CM DEPTH FROM THE NOSE AND MIX IT INTO THE POST-SADIUM EXTRACTION SOLUTION