

**NOTE:**

Each test is disposable. Do not try to use the test more than once.

**BNG® COVID-19 Neutralizing Antibody Test Kit (Colloidal Gold)**


**Test Procedure**




1) Take out the test cassette, cleaning tissue, needle and solution from the box.




2) Wear the gloves. Use a new glove for each patient.

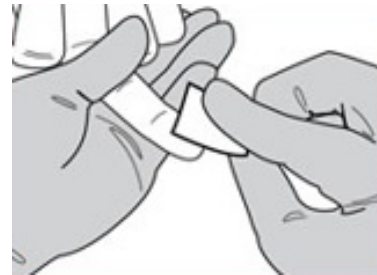

3) Wipe your finger with the cleaning tissue.

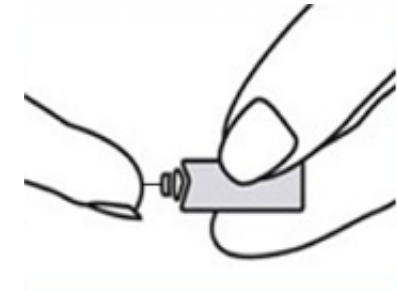


4) Unscrew the cap of the sterile, disposable, blue colored needle and pierce the fingertip.

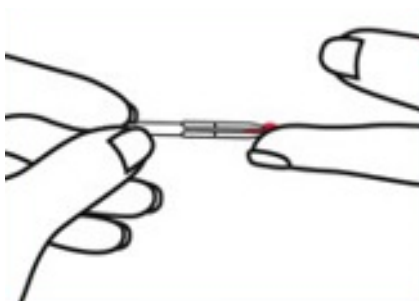

5) Collect approximately 10  $\mu$ L of blood sample with a micropipette.



7) Transfer the blood sample into the sample well (S) of the test cassette. Drop 3 drops of the buffer solution on it. Wait 15 minutes. Do not interpret results after 30 minutes.

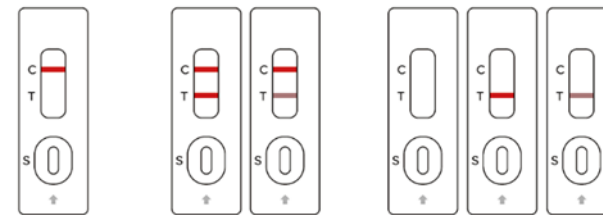

Negative

Positive

Invalid

**Positive:** Two lines should be identified. One of the colored lines should be in the control line region (C) and the other line should be in the test region (T).

**Negative:** Only the control wise (C) should have a single colored line. No lines should be visible in the (T) region.

**Invalid:** If the control line (C) is not visible, the test is invalid.